


SECURE[®] FLEXIBLE BRACKET

Model # FB-1 – For Use With Secure PIR Motion Detection Alarm (PIR-1)

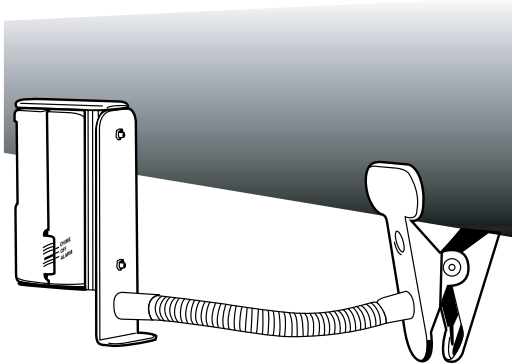

Contents of package:

- Flexible bracket
- Two screws with nuts to attach PIR alarm to bracket

TO ATTACH PIR ALARM TO BRACKET:

1. Remove PIR alarm from plastic alarm holder attached to alarm.
2. Line up holes on back of alarm holder to holes on bracket base.
3. Affix alarm holder to bracket base using the two screws included.
Tighten nuts on screws to ensure alarm is securely affixed to bracket base.
5. Replace PIR alarm into plastic alarm holder. Attach clamp at end of bracket to bed at foot of bed with PIR alarm lens directed towards a wall.

Note: The bracket has an extra long neck to prevent blankets from obstructing lens of PIR alarm. Make certain blankets do not obstruct lens.

Refer to PIR alarm instructions contained in alarm box for alarm operations instructions.

WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from date of purchase.

CAUTION!

This product is not a restraint. It will not prevent a resident from getting out of their bed or chair. This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair. This product is to be used in conjunction with a total fall prevention program

Please see our complete line of Secure[®] fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Cedar Rapids, Iowa 52402 • 800.3.SECURE