

Alert-Mate[®] ALARM

Restraint-Free Fall and Wandering Prevention Alarm

Model # SAM-1 • Made in Taiwan



**One-year
Replacement
Warranty**

**Locking
Clip**

NEW!
 Choice of top
 or front pin
 positions to
 activate alarm
 from any angle for
 increased resident
 protection

Size: 2-1/4" x 2-3/4" x 1"

U.S. Patent No. 6,239,704

Created by and for long-term care professionals. Used in thousands of concerned care facilities across the U.S.


- OSHA approved 80 decibel alarm level
- 4' adjustable cord with locking clip
- 9-volt battery included
- Battery tester
- VELCRO[®] attachment

TO USE: If using Velcro[®] strip on back of alarm, remove paper backing and affix alarm to bed, chair, wheelchair or nightstand out of reach of resident. If using one of the Alert Mate[®] Holders, AMH-1 or AMH-2 (sold separately), insert alarm into holder and attach to bed, chair, wheelchair or nightstand out of reach of resident. Insert pin at end of cord in either front or top pin receptacle of Secure[®] Alert-Mate[®] Alarm. Adjust cord to desired length. Attach cord to resident's clothing with locking clip so that the clip is not accessible to the resident. Be sure to gather adequate clothing material in clip before closing for best holding results. Either pre-test with a staff member or have the resident stand up to ensure pin will release from alarm when the user stands up. If resident moves beyond desired range, pin will come out and alarm will alert staff. Test alarm each time it is used to make sure it functions properly.

**ASK YOUR ACCOUNT SERVICE MANAGER ABOUT
 THE ALERT-MATE[®] MOVEABLE ALARM HOLDER.**

TO INSTALL BATTERY

1. Loosen the two screws on back of alarm with a small Phillips (included) or slotted screwdriver
2. Remove back of alarm
3. Install 9-volt battery as indicated
4. Replace back panel and tighten screws

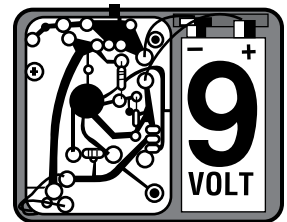

WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

CAUTION!

This product is not a restraint, therefore will not prevent a resident from getting out of their bed or chair. This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair. This product is to be used in conjunction with a total fall prevention program.

Please see our complete line of Secure[®] fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.3.SECURE