

Loud-Door Resident Safety Alarm

Model # SLD-1

Created for health care professionals to alert staff to potential wandering or fall situations. Use doorknob cord to hang alarm on resident's room and/or bathroom door. Alarm can also be mounted on doorframe using Velcro® strip or mounting bracket and screws included.

The **Secure® Loud-Door Resident Safety Alarm** was designed to assist health care professionals in preventing wandering from rooms and falls in bathrooms. The 80 decibel alarm will alert staff without frightening resident.

OPERATING INSTRUCTIONS

1. Insert metal pin at end of cord into opening at top of alarm. Keep metal pin inserted into alarm at all times.
2. Install 9-volt battery (included) per the "To Install Battery" instructions

Please note: If you install the battery prior to **Step 1** the alarm will sound continuously until **Step 1** is completed. When the battery is installed the alarm will sound continuously anytime the metal pin is removed from the alarm or the prongs are separated.

3. Slip doorknob cord over doorknob to put alarm in position. If you would prefer to mount the alarm to a doorframe you can either use the Velcro® strip or the mounting bracket and screws included in the package.
4. Holding the two prongs at the end of the cord together, remove the plastic holder. To store the plastic holder slip it over one of the prongs. While holding the prongs together insert the prongs between the door and the doorframe. Any

Door Clip

- Holding the two prongs of door clip together, remove plastic holder and insert prongs between door and door frame.


time the prongs are separated the alarm will sound.

Please note: If you should lose the plastic holder use a small rubber band to hold the prongs together when alarm is not in use.

5. When the door is opened the two prongs will fall out and separate causing the alarm to sound continuously.
6. To shut the alarm sound off push the two prongs together. The alarm will stop. If you want to continue using the alarm close the door and insert the prongs between the door and the doorframe once again.
7. To store the alarm, remove the plastic piece from the one prong and slip it over both prongs to hold them together preventing the alarm from sounding. Always keep the metal pin inserted in the top of the alarm during storage.

SAFETY TIPS

1. Place **Secure® Loud-Door Resident Safety Alarm** on resident's entry and bathroom doors to prevent potential wandering from room or falls in bathrooms.
2. Test alarm each time prior to use.

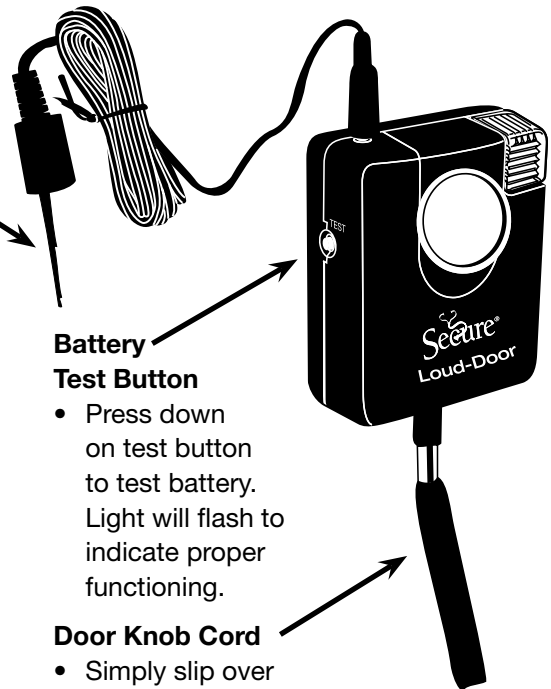

Battery Test Button

- Press down on test button to test battery. Light will flash to indicate proper functioning.

Door Knob Cord


- Simply slip over door knob to put alarm in position
- Velcro® strip included to attach alarm to doorframe as desired.

WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from date of purchase.

TO INSTALL BATTERY

1. Loosen the two screws on back of alarm with a small Phillips or slotted screwdriver
2. Remove back of alarm
3. Install 9-volt battery as indicated
4. Replace back panel and tighten screws

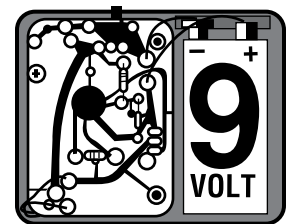

Please see our complete line of Secure® fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Cedar Rapids, Iowa 52402 • 800.3.SECURE