

SECURE® MAT ALARM-120

Resident Safety Alarms For Use In Rooms Farthest From The Nurse's Station

To be used **ONLY** with Secure® Reversible Floor Mats
(MAT-1 – 36"x24" or MAT-2 – 60"x24")

Model # SMA-120 • Made in China

The Secure® Mat Alarm-120 is a tamper-resistant alarm created for nursing homes and hospitals that require an alarm sound decibel level up to 120 decibels. This alarm is ideal for care facilities with resident rooms toward the end of long hallways. In addition, the option of setting the decibel level at 120, 90 or 80 decibels enables staff to determine an appropriate sound level for several different room locations.

The alarm features:

- Always "ON" for resident safety.
- Tamper-Resistant.
- Flashing alert light will instantly identify roommate in need... day or night.
- Choice of 120, 90 or 80 decibels.
- Three AAA batteries included.
- Low battery indicator light.
- Test & reset button.
- Alarm holder.
- AC/DC adapter option (sold separately).

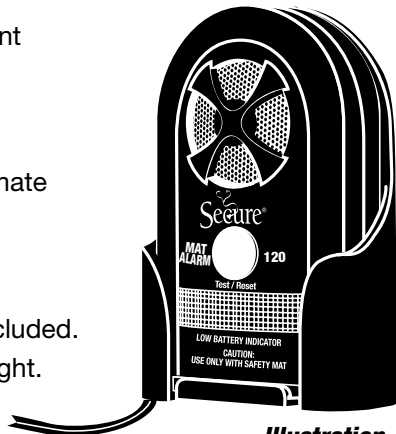


Illustration A

INSTRUCTIONS

1. **WARNING to avoid possible injury:**

- Carefully read and understand the Secure® Reversible Safety Mat instructions prior to use.
 - NEVER connect the Secure® Reversible Safety Mat Alarm to a bed or chair pad. The Secure® Reversible Safety Mat Alarm's function is exactly the reverse of a bed or chair pad's function. The safety mat system will sound when pressure is applied to the mat. A bed or chair pad system will sound when pressure is released from the pad.
2. Install three AAA batteries (included) into alarm by opening back cover with flathead screwdriver. The alarm will beep twice. The alarm also has an AC adapter plug-in option

on the side of the alarm. Please refer to Illustration A. If at all possible it is recommended that you use an AC/DC Adapter along with the batteries to prolong the life of the batteries. (See our order form to order our Secure® 6-Volt AC/DC Adapter - Model # ACDC-1).

3. You will need to choose the decibel level that best suits your situation. You can test each decibel level by moving the "120/90/80 dB" switch to each decibel level and pressing and holding the "Test/Reset" button on the front of the alarm. After decibel is chosen replace the battery cover and tighten tamper-resistant screw.
4. Remove tamper-resistant screw from sliding cover on side of alarm. Insert metal pin of the floor mat connecting wire into opening marked "Sensor Mat."

Test Secure® Mat Alarm and Secure® Reversible Floor Mat by stepping on mat. The alarm should sound when pressure is applied to the mat. To turn alarm off simply press the "Test/Reset" button 3 times quickly in 1.5 seconds.

Push connecting wire down into slot opening at the end of alarm. Replace tamper-proof cover and tighten screw.

CAUTION! ALWAYS remove sensor mat pin from alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.

5. You have now completed all the alarm function set up steps.
6. Remove the paper covering from the top of the Velcro® piece attached to the bottom of the alarm. Slide the alarm into the alarm holder carefully threading the mat cord through the slot opening at the bottom of the holder. Press down on the top of the alarm while pulling the cords through the opening to secure the Velcro® piece to the bottom of the holder. The alarm is now firmly in the holder. Please refer to Illustration A. When it is necessary to remove the alarm from the holder simply pull up on the alarm to separate the Velcro® pieces. Remove the wires by sliding them through the slot at bottom of alarm holder to completely remove the alarm from the holder.

Continued

Please see our complete line of Secure® fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Cedar Rapids, Iowa 52402 • 800.3.SECURE

SECURE® MAT ALARM-120

Resident Safety Alarms For Use In Rooms Farthest From The Nurse's Station

Continued

7. Simply wrap the Velcro® strap with the rubberized non-slip backing attached to the back of the holder tightly around railing or frame of bed or nightstand. Holder placement should be easily accessible to staff but out of reach of the resident. Secure excess cord to avoid damaging cord.
8. **The alarm and floor mat should be checked prior to each use for proper functioning. Test the alarm by stepping on the mat. The alarm will sound when you step on the mat. The alarm will continue to sound and the flashing light will continue to flash for resident safety. To turn the alarm off simply press the "Test/Reset" button 3 times quickly in 1.5 seconds. You can also test the alarm by pressing the "Test/Reset" button. DO NOT ASSUME! TEST EACH TIME THE SYSTEM IS USED.**
9. The light bar on the front of the alarm also functions as a low battery indicator. When the batteries are low and require changing the light bar will flash blue intermittently and the alarm will emit a beeping sound. Note: As a safeguard simply pressing the reset button will not stop the alarm from beeping when the batteries are low. The blue light will continue to flash and the beeping sound will continue until the batteries are changed.

12 MONTH LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of 12 months from the date of purchase. Void if this product has been misused, abused or punctured. No warranty of responsibility of use expressed or implied.


Please note: For resident safety, the Mat Alarm-120 does not have an "On/Off" switch. Staff members do not need to remember to turn alarm back on as alarm has an auto reset feature.

CAUTION!


This product is not a restraint. It will not prevent a resident from getting out of their bed or chair.
This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair.
This product is to be used in conjunction with a total fall prevention program.



Secure® Hip Protectors - SHP
(In five sizes & 3 colors)

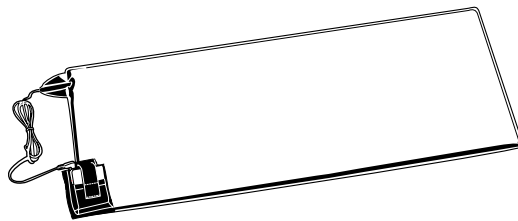

Secure® Bed Pads-1 Set

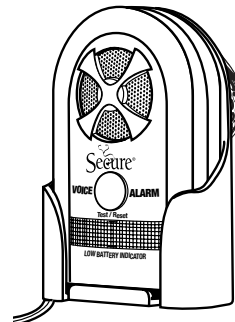

**Secure® Voice Alarm
SVA-1**



**Secure® Universal Alarm
SUA-1**

Please see our complete line of Secure® fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Cedar Rapids, Iowa 52402 • 800.3.SECURE