

# OPEN FRONT HIP PROTECTOR

## With Removable Pads

Model # SOHP • Made in China

### Instructions

Secure® Hip Protectors are to be used as part of a total fall management program. Please note that no hip protector can prevent all injuries. The Secure® Hip Protector's strategically positioned high impact absorbing foam pads will greatly assist in optimizing the potential to prevent serious injuries including fractured hips as a result of a fall. Please see independent hip protector research results at [www.padalarm.com](http://www.padalarm.com).

The Secure® Hip Protector easily fits over underwear and has adjustable Velcro® leg closures for maximum leg comfort. The garment is made from quick drying, Latex-free, 100% Terylene. The unique memory foam pads will conform to the user's body contours. The memory foam pads will be undetectable when worn with regular, loose fitting clothing. Please note it may take some users one or two days to acclimate to wearing the protective garment.

Using a permanent ink pen write the user's name, room number and date of first use on the garment label and the two removable pad labels. Insert pads securely inside pockets of garment. Open adjustable Velcro® waistband and place garment around user's waist at navel level. Adjust waistband for maximum user comfort. Securely fasten waistband's Velcro® closures together. Carefully adjust the hip protector's pads to make certain the pads adequately cover both hips of the user. Adjust the Velcro® leg bands around each leg for maximum user comfort and securely fasten the Velcro® closures together.

### Washing Instructions

According to the CDC's Guideline for Laundry in Health Care Facilities found on their Web site ([www.cdc.gov/od/ohs/biosfty/laundry.htm](http://www.cdc.gov/od/ohs/biosfty/laundry.htm)), "Hot water provides an effective means of destroying microorganisms." The guidelines further state: "Rather than rigid rules and regulations, hygienic and common-sense storage and processing of clean and soiled linen is recommended."

Please note that the removable high impact foam pads of the Secure® Hip Protectors are waterproof sealed. **FOR BEST RESULTS PLEASE REMOVE PADS WHEN WASHING OR DRYING THIS GARMENT.** If the pads are removed prior to washing, please clean with a mild, liquid disinfectant prior to inserting pads in pockets. Never puncture or cut the waterproofed seal of the foam pads or the foam pads will be damaged. The garment can no longer be used and the 6-month limited warranty will be void. See 6-month warranty details below.

We recommend washing the garment in water with a temperature of at least 104°F (20°C). Dry at medium temperatures. Please note that drying at lower temperatures will prolong the life of the garment. Chlorine bleach provides an extra margin of safety according to the CDC. Bleach as directed on your bleach container. **DO NOT IRON.**

### Sizes Available:


Model #	Size	Fits Waist Sizes
SOHP-S/M	Small/Medium	28"-34"
SOHP-L/XL	Large/X-Large	34"-42"

### 6 Month Limited Warranty

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of 6 months from date of first use. Void if this product's waterproofed sealed foam pads have been misused, abused or punctured. No warranty or responsibility of use expressed or implied.

### Replacement Pads Available (SHP-PAD)

**\$1.99 each**

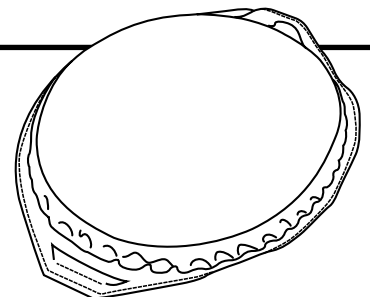

## CAUTION!

There is no guarantee that the user will be protected against all injuries as a result of a fall. Please note in your procedures for using this product that the user's skin should be checked regularly. The garment must be washed after each incontinent event to protect against skin rash.

Please see our complete line of Secure® fall & wandering prevention products at [www.padalarm.com](http://www.padalarm.com).

**Highest Quality...Lowest Prices...Guaranteed!**

**Personal Safety Corporation • Cedar Rapids, Iowa 52402 • 800.3.SECURE**