

FALL PREVENTION WITHOUT THE NOISE!®



SECURE® SILENT MONITOR WITH HOLDER & FREE \$4.99 VALUE NURSE CALL CABLE (SSM-1)



PSC recommends using this product with Secure® Bed Pads only

INSTRUCTIONS

The Secure® Silent Monitor With Holder & Nurse Call Cable was designed to completely eliminate the ambient noise in a resident's room while providing maximum fall management benefits. The monitor and 8' nurse call cable can be used with our Secure® Bed Sensor Pad, Secure® Chair Sensor Pad or the Secure® Seat Belt. If using with the Secure® Seat Belt please refer to the seat belt instructions. The following instructions cover the use of this product with the Secure® Bed or Chair Pads. **PLEASE NOTE: THIS PRODUCT IS NOT A RESTRAINT. IT WILL NOT PREVENT A RESIDENT FROM GETTING OUT OF THEIR BED OR CHAIR.**

Please refer to Illustration A below for the following steps:

1. Open the sliding tamper-resistant cover on side of alarm with a screwdriver.
2. Insert nurse call plug into opening marked "Nurse Call." Insert pin of bed, chair or seat belt into opening marked "Sensor Pad."

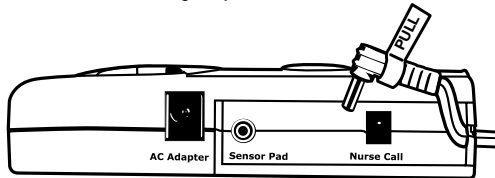


Illustration A

CAUTION! ALWAYS remove sensor pad pin from alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.

3. Carefully thread the nurse call cord and the pad or seat belt cord through the slot opening at the bottom of the alarm.
4. Replace the sliding tamper-resistant cover and tighten screw with screwdriver to prevent tampering.

Please refer to Illustration B below for the following steps:

5. Open battery cover on back of alarm by removing tamper-resistant screw and sliding the battery cover in the direction of the arrow with your thumbs. The battery cover will open and stay attached to the monitor. Install three "AAA" batteries (included) into the battery compartment per the (+/-) diagrams on the bottom of the battery holder.
6. The monitor also has an AC adapter plug-in option on the side of the monitor.

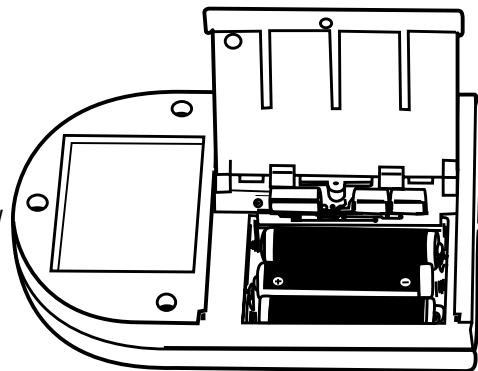


Illustration B

Please refer to Illustration A above. If at all possible it is recommended that you use an AC/DC Adapter along with the batteries to prolong the life of the batteries. (It is recommended that you use our Secure® 6-Volt AC/DC Adapter

to avoid possible damage to the alarm. See our order form to order our Secure 6-Volt AC/DC Adapter – Model # AC/DC-1)

7. You have now completed all the monitor function set up steps. Close the battery cover and replace the tamper-resistant screw.
8. Remove the paper covering from the top of the Velcro® piece attached to the bottom of the monitor. Slide the monitor into the holder (included) carefully threading the pad and nurse call cords through the slot opening at the bottom of the holder. Press down firmly on the top of the monitor while pulling the cords through the slot opening at the bottom of the holder. The monitor is now firmly in the holder. When it is necessary to remove the monitor from the holder simply pull up on the monitor to separate the Velcro® pieces. Remove the cords by sliding them through the slot at bottom of monitor holder to completely remove the monitor from the holder.
9. Affix the alarm and holder securely to the side of a bed or chair out of sight and reach of resident. Simply wrap the Velcro® strap with the rubberized non-slip backing attached to the back of the holder tightly around railing or frame of bed or wheelchair. Holder placement should be easily accessible to staff. Secure excess cord to avoid damaging cord.
10. Carefully assist resident into bed or wheelchair positioning resident's buttocks in center of pad.
11. Push "Test/Reset" button on front of monitor to test. The light bar will flash red intermittently to indicate monitor is functioning properly. The monitor will allow for normal movement. However, if resident gets off sensor pad the monitor will alert staff at the nurse's station. When the resident gets back on the pad the monitor will automatically reset. **However, the light bar will continue to flash red intermittently to let staff members know which resident activated the monitor. The staff member needs to press the reset button to shut off the flashing red light. The monitor is then automatically reset.**
12. When staff members remove the resident from the pad for whatever reason the monitor will alert staff at the nurse's station. When the resident is positioned back on the pad the red flashing light will continue until a staff member presses the monitor's "Test/Reset" button. The monitor is then set for the next exit event.
13. The monitor and sensor pad should be checked prior to each use for proper functioning. **DO NOT ASSUME! TEST EACH TIME THE SYSTEM IS USED.**
14. The light bar on the front of the monitor also functions as a low battery indicator. When the batteries are low and require changing the light bar will flash **BLUE** intermittently. Note: As a safeguard the flashing blue light will continue until the batteries are changed.

12 MONTH LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from date of purchase. Void if the product is misused, abused, or punctured. No warranty or responsibility of use expressed or implied.

Please see our complete line of Secure® fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Cedar Rapids, Iowa 52402 • 800.3.SECURE