

FALL MANAGEMENT BLANKET

Model # SFMB-1Y • SIZE: 30"L x 48"W • Made in China

PURPOSE:

This **Secure® Fall Management Blanket** is part of our total yellow fall management program. Yellow is the generally accepted color for fall risk residents and patients.

All of the **Secure®** yellow fall management products, to include this blanket, are designed to preserve the dignity of high fall risk residents and patients while at the same time immediately identifying the user to staff members as a fall risk individual. The **Secure® Fall Management Blanket** is made from a proprietary blend of the softest polyester fiber materials possible to provide comfort and warmth for your residents and patients. The blanket has been tested and meets the CA #117 Section E Flame Retardancy Standards. The CA #117 test results can be printed from our website at: www.padalarm.com/images/TestingDocs/SFMB-1Y-CA117-FlammabilityTestResults.pdf.

HOW TO USE:

The **Secure® Fall Management Blanket** can be used as a resident or patient's lap blanket to provide comfort and warmth while also serving as an immediate staff visual alert to a fall risk individual. The blanket can also be placed across the foot of patient or resident's bed or on the back of an in-room chair. In either in-room use, the blanket will provide an immediate fall risk visual cue to all staff members entering the room.

This blanket is for indoor use or to provide warmth on outdoor wheelchair walks. To extend the life of the blanket please store in your facility's storage or stock room under regular facility temperatures and humidity levels. Avoid excessive high humidity levels that can create excessive moisture conditions.

CLEANING INSTRUCTIONS:

Always wash this blanket with items of a similar light color. The blanket should be machine washed in lukewarm water (100°F, 38°C). If bleach is used, please follow the directions



on the bleach container. Always dry on low temperature to extend the life of the blanket.

WARRANTY

Personal Safety Corporation warrants this blanket to be free from factory defects in materials and workmanship for a period of **SIX MONTHS** from the date of purchase. Warranty is void if this blanket is misused, abused, or damaged. No warranty or responsibility of use expressed or implied.

WARNING!

Please dispose of this blanket according to your facility's **BIOHAZARDOUS MATERIALS POLICY**.

Visit www.padalarm.com for info on our **Secure® Fall Prevention Admission Kit**



Please see our complete line of **Secure® fall & wandering prevention products** at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.3.SECURE (800.373.2873)