

UNIVERSAL ALARM

For Use as a Pad Alarm, Seat Belt Alarm or Personal Alarm

Model # SUA-1 & SUA-1 Yellow • Made in Taiwan

**Use
With!**

**Bed Pad • Chair Pad • Under Mattress Pad • Toilet Seat Pad
or as a Personal Alarm**

The Secure® Universal Alarm was created by and for long-term care professionals. This alarm is currently used in thousands of concerned care facilities across the U.S. The alarm features:

- Always "ON" for resident safety.
- Flashing alert light will instantly identify roommate in need...day or night.
- OSHA approved 80-decibel sound level.
- Alerts staff without frightening resident.
- Two AAA batteries included.
- Low battery indicator light.
- Test & reset button.
- Velcro® attachment.



INSTRUCTIONS

1. Install two AAA batteries (included) in alarm by opening back cover with Phillips screwdriver. Replace battery cover and tighten screws. **PLEASE NOTE: Alarm will immediately sound once batteries are installed. To silence the alarm press "Test/Reset" button on side of alarm 3 times within 2 seconds.**
2. Affix Secure® Alarm Holder (AMH-1) to bottom or back of bed or chair out of sight and reach of resident. Simply wrap Velcro® strap with rubberized non-slip backing around railing or frame of bed or chair. **CAUTION: Do not affix alarm to a PVC rail. The strap may permanently stain the rail.** Holder placement should be easily accessible to staff.
Note: If not using an alarm holder remove the paper backing of Velcro® strip attached to back of alarm. Affix the alarm to bed, chair, wheelchair or nightstand using the adhesive backing of the Velcro® strip.
3. If using with Bed Pad: see instructions printed on Bed Pad.
If using with Chair Pad: see instructions printed on Chair Pad.
If using with Toilet Pad: see instructions printed on Toilet Pad.
If using with Under the Mattress Pad: see instructions printed on Under Mattress Pad.
If using with Seat Belt: see instruction sheet included with Seat Belt.
4. Insert metal pin of pad/seat belt being used into opening at top of alarm. **CAUTION! ALWAYS remove sensor pad pin from alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove**

pin any other way or you may damage pin and void warranty.

5. Put alarm in holder and fasten Velcro® closure of holder to secure alarm in holder. Secure excess cord of sensor pad to avoid damage.
6. Carefully assist resident onto pad as explained on pad or into seat belt as explained on instruction sheet included with seat belt.
7. Push "Test/Reset" button on side of alarm. The alarm will sound four times, stop sounding and automatically reset to sound in the event of an exit attempt. Alarm will allow for normal movement. However, if resident gets off pad the alarm will sound to alert staff. When resident gets back on pad the alarm will automatically reset.
8. When staff removes user from pad for whatever reason the alarm will sound. **To silence the alarm press "Test/Reset" button on side of alarm 3 times within 2 seconds.** When the resident is positioned back on the pad the alarm will automatically reset to the alarm mode. **Please note: For resident safety, the alarm does not have an "On/Off" switch. Staff members do not need to remember to turn the alarm back on as the alarm has an auto reset feature.**
9. The alarm must be checked prior to each use for proper functioning. The "Test/Reset" button also functions as a battery tester. In addition the Low Battery Light on the front of the alarm will light up when it is time to change the batteries.

Please note: If using as a personal alarm with Pin/Cord & Clip, adjust cord to desired length. Insert pin at end of cord into opening at top of alarm. Attach cord to resident's clothing with locking clip. When resident moves beyond desired range, pin will come out and alarm will alert staff.

WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from the date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

CAUTION!

This product is not a restraint. It will not prevent a resident from getting out of their bed or chair.
This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair.
This product is to be used in conjunction with a total fall prevention program

Please see our complete line of Secure® fall & wandering prevention products at www.padalarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.3.SECURE