

# WEDGE SEAT CUSHION WITH SAFETY STRAPS

**Size: 18" W x 16" L**

**Model # SWSC-1 • Made in China • One Year Warranty**

The Secure® Wedge Seat Cushion was designed to provide proper positioning & comfort for the resident. The tapered 4" to 2" design promotes anterior tilt for pressure-relief. The ultra soft high-density foam provides long-term care residents seating comfort. Designed for wheelchairs and geri-chairs

**NOTE:** Meets California #117 Flame Retardant Standards

## UNIQUE FEATURES AND BENEFITS

- High-Density Foam For Resident Comfort
- Moisture & Mildew-Resistant Vinyl
- Slip-Resistant Bottom Prevents Movement In Chair
- Safety Straps To Secure Seat Cushion Placement
- Outer Cover Can Be Unzipped Quickly For Cleaning
- Ultra Soft Vinyl
- Easy To Clean
- Latex Free



Photo A



Photo B

## HOW TO USE

Separate yellow Velcro® safety straps attached to the end of the wedge seat cushion. Position Secure® Wedge Seat Cushion onto wheelchair seat with slip-resistant surface facing down. The end with safety straps will be facing towards back of chair. (See Photo A). Slide safety straps through opening between seat back and wheelchair frame on both sides of chair. Affix Velcro® straps together to secure seat cushion in place. (See Photo B). Carefully assist resident into wheelchair.

**CLEANING INSTRUCTIONS:** Wipe clean with a damp cloth using mild soap or liquid disinfectant as needed.

**ONE YEAR WARRANTY:** Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if product's outer covering is misused, abused, cut or punctured. No warranty or responsibility to use expressed or implied.

## CAUTION!

This product is not a restraint. It will not prevent a resident from getting out of a wheelchair. Please properly position the Secure® Wedge Seat Cushion according to the instructions above with each use to provide maximum support and comfort for the resident.

Please see our complete line of Secure® fall & wound care management products at [www.PadAlarm.com](http://www.PadAlarm.com).



Secure® Wedge Pommel Cushion



Secure® Lap Cushion



Secure® Deluxe Arm Support

Please see our complete line of Secure® fall & wandering prevention products at [www.PadAlarm.com](http://www.PadAlarm.com).

**Highest Quality...Lowest Prices...Guaranteed!**

**Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873**